INSTRUCTION FOR USE

GHIACCIO ISTANTANEO MONOUSO

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Rev. 12 - 04/2024

ISTRUZIONI

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Rev. 12 - 04/2024

INSTRUCTIONS FOR USE

Break the inside water bubble, by squeezing the bag with both hands. Then shake it in order to mix the contents. Apply on the interested part.

INDICATIONS

Haematomas, sprains, injuries, hyperthermia and whenever cold application is required. The cold pack has a temporary anaesthetic and vasoconstrictor effect.

CAUTION

For external use only; disposable product. The product no longer works after being used. Do not use if the bag is damaged. Do not break the bag and by all means prevent any content leakage. The content of the bag causes stains and irritation (in case of contact, rinse with water for 15 minutes), and is harmful (if swallowed seek immediate medical assistance). Store at room temperature out of the reach of children and away from heat sources and flammable materials. Do not freeze the bag, do not use on injured skin. Contact your doctor in case of cardiovascular diseases, circulatory diseases, diabetes and/or cutaneous insensitivity (Raynaud's syndrome), if the product is used on children and the elderly, or in case of doubt on how to perform the treatment and if the symptoms persist. Do not use on babies and children under the age of 3. In the event of any serious safety incident occurring by using the medical device, please report it to the Manufacturer and to the Competent Authority of your Country.

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