INSTRUCTION FOR USE

CALDO ISTANTANEO MONOUSO

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Rev. 10 - 07/2024

ISTRUZIONI

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Rev. 10 - 07/2024

INSTRUCTIONS FOR USE

Break the inside water bubble, by squeezing the bag with both hands. Then shake it in order to mix the contents. Apply on the interested part.

INDICATIONS

Lumbago, stiff-neck, rheumatism, hypothermia and whenever hot application is required. The hot pack provides relief in the treated area .

CAUTION

For external use only; disposable product. The product no longer works after being used. Do not use if the bag is damaged. Do not break the bag and by all means prevent any content leakage. The content of the bag causes stains. If ingested the contents are noxious. In that case consult a doctor immediately. Store at room temperature out of the reach of children and away from heat sources and flammable materials. Do not use on injured skin. Do not use on babies and children under the age of 3. Contact your doctor in case of cardiovascular diseases, circulatory diseases, diabetes and/or cutaneous insensitivity (Raynaud's syndrome), if the product is used on children and the elderly; in these cases wrap the bag with a towel. Contact your doctor in case of doubt on how to perform the treatment and if the symptoms persist. In the event of any serious safety incident occurring by using the medical device, please report it to the Manufacturer and to the Competent Authority of your Country.

CONTENT

Anhydrous magnesium sulphate and water .

Dispotech Srl - Via al piano 29, 23020 Gordona (SO)
Italy - www.dispotech.com - e-mail info@dispotech.it
Tel. +39 0343/36711 - Fax. +39 0343/36567

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